

myride User Manual

A more
comfortable
to travel !!



Mechanical brake

Loosen the mechanical brake of the rear wheel, and you can implement the smart scooter.



↑ electric



↓ manual

Turn on/off



ON

Press  button about 3 seconds.



Press  button.



OFF

Press  button about 3 seconds.



Ride and leave

※Check if the mechanical brake is in locked state.

01 Rotate the handrail upward the position.



02 From the side sit on the seat cushion.



03 Adjust the sitting position. And lower the handrail.



04 Fasten your seat belt.



Tips for safe seating




- Do not seat from the front.
- The seating should not sit backwards, and should not stand on the seat.
- Do not ride by two people.



Drive


Advance

Bring the smart scooter forward by pushing the joystick.

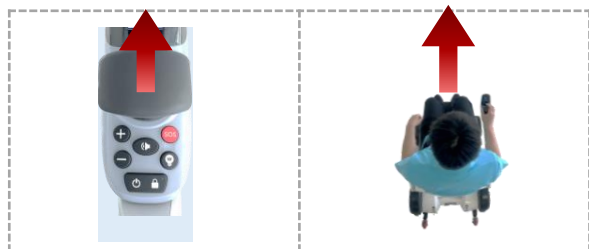


Draw back

By pulling the joystick, the smart scooter will step back.

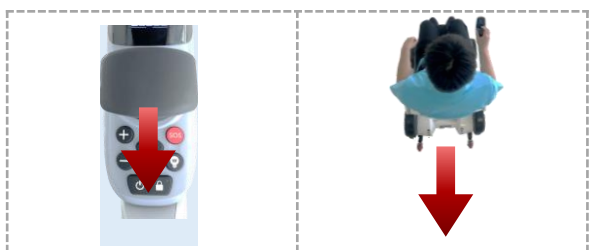


Turn left direction

By moving the joystick left to make the smart scooter rotate left in place.

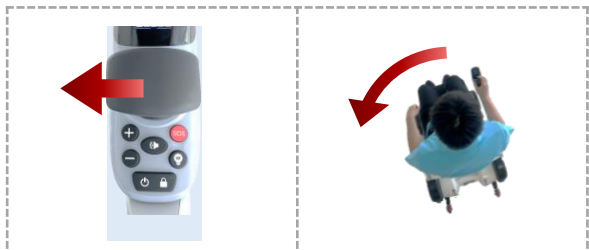


Turn right direction

By moving the joystick right to make the smart scooter rotate right in place.

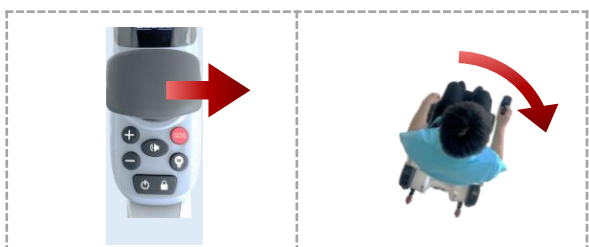

Display

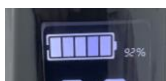


Represents the percentage of the battery.



Represents the miles per hour.



The meter represent the speed gear .

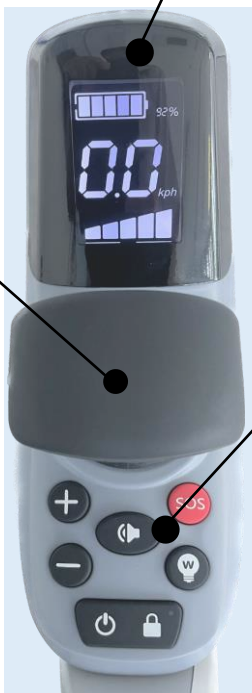

Key function



Plus button :
Raise the speed range.



Minus button :
Lower the speed range.

※The driving speed of the intelligent mobility scooter is 5 gears..



Horn button :
Honk the horn.



SOS button :
Calls for help.



Mode witching button :
Beginner mode for novice operation or standard mode for skilled operation is switched.

Folding function

Fold

At the same time, press and hold button for about 3 seconds.



expand

At the same time, press and hold button for about 3 seconds.



Vertically



Tips for safe driving

- Electric carts are wheelchairs for people with physical disabilities under the Road Traffic Act and are treated as pedestrians. Please observe traffic rules and manners as a pedestrian.
 - Do not use for any purpose other than an electric cart.
 - Never use mobile phones or wireless devices while driving.
- When using then, stop the electric cart in a safe place and turn off the power switch.
- Avoid driving in the following places or situations.
- ※Crowd, Busy roads, Bumpy tiles for blind

